



in2it™ A1C

Be the Difference
With Fast, Accurate Results






5 Good Reasons to Use in2it™

1 Improved patient care

- Studies show that discussing A1C results with patients in real-time, significantly improves glycemic control. A1C levels were reduced up to 1.44% HbA_{1c} after 6 months and up to 1.65% HbA_{1c} after 12 months in the immediate assay group compared to the control group.*
- in2it provides A1C information needed to effectively improve glycemic control in type 1 and insulin-treated type 2 diabetic patients.


* E. Cagliero et al: Immediate Feedback of HbA_{1c} Levels Improves Glycemic Control in Type 1 and Insulin-Treated Type 2 Diabetic Patients.

Diabetes Care, Vol. 22, No. 11, November 1999.

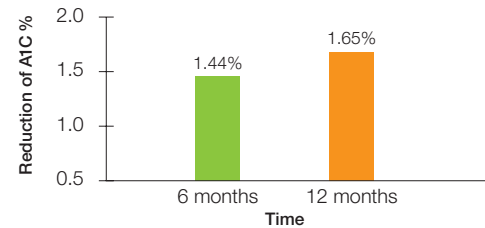

2 in2it™ is easy to use

The in2it™ is **CLIA waived**¹, fully automated and easy to use. in2it™ results are obtained using a 3-step process simplifying instrument operation and staff training.

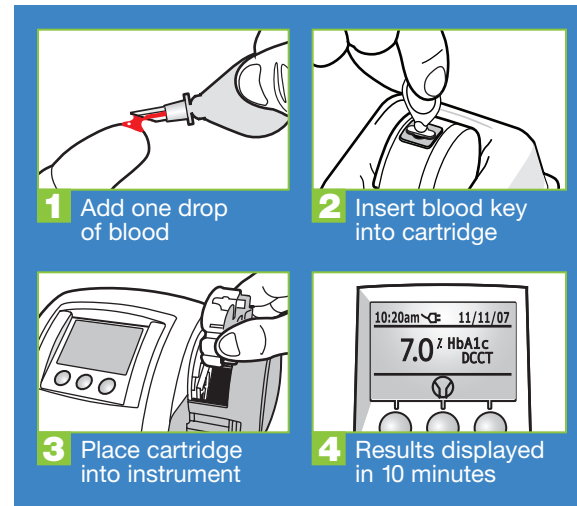

¹ 510(k) cleared for prescription home use and, accordingly, is CLIA waived.

3 in2it™ delivers reliable results

The in2it™ correlates closely to established HPLC laboratory methods. In studies using HPLC instruments, the correlation values were accurate and reproducible. This level of correlation ensures that the results generated in your office will closely match laboratory results.

Reproducible

Laboratory precision produces consistent results with minimal variation.

in2it™ Result	Normal	Diabetic	High Diabetic
First	5.1	7.1	9.2
Second	5.0	7.1	9.1
Third	5.1	7.0	9.3

Accurate

Result comparison with NGSP reference values assigned using HPLC.

	Normal	Diabetic	High Diabetic
NGSP % HbA _{1c}	5.6	8.0	10.9
in2it™ Result % HbA _{1c}	5.8	7.8	10.8

(average of seven samples per range)

4 in2it™ improves workflow

- Each in2it™ system includes a keypad and printer to ensure that patient identification is not compromised.
- in2it™ stores the previous 200 patient results and can be downloaded using the USB port.
- The adhesive backed reports provide permanent storage and eliminate transcription errors.



Keypad

Printer

Print out

5 in2it™ results are reimbursable

- The A1C CPT code is 83037QW. The national Medicare reimbursement limit is \$14.17 per test.
- in2it™ reduces costs by eliminating the need for A1C send-out testing.

in2it™ A1C

Better patient care at your fingertips

- Easy, fully automated operation simplifies training and use
- Extremely small footprint requires minimal workspace
- Same-visit results facilitate patient education and consultations, improving patient compliance
- in2it™'s CLIA waiver* allows your existing staff to perform the test
- Reimbursable A1C for tests performed in your office

Bio-Rad – The world leader in A1C testing

For over 30 years, Bio-Rad has been the gold standard in A1C testing. Today, over 4,000 hospitals and reference laboratories worldwide choose Bio-Rad A1c because of Bio-Rad's superior quality and service.

The in2it™ brings Bio-Rad A1c to your patients while they wait. Rely on Bio-Rad to help you be the difference for every diabetic patient.

Ordering Information

Catalog No.	Description	
281-0000	CLIA Waived in2it™ Analyzer	1 unit
281-0001	in2it™ A1C Test Cartridges	10 tests
281-0002	in2it™ System Check Cartridge	1 unit
281-0003	in2it™ A1C Quality Controls	1 unit
281-0004	in2it™ Printer.....	1 unit
281-0006	in2it™ Keypad.....	1 unit
281-0007	in2it™ Bar Code Reader.....	1 unit

* 510(k) cleared for prescription home use and, accordingly, is CLIA waived.



**Bio-Rad
Laboratories**

*For further information and/or to view an in2it video,
please visit www.in2itdemo.com*

*Clinical
Diagnostics Group*

Website www.bio-rad.com/diagnostics **U.S.** 1-800-2BIO-RAD **Australia** 61-2-9914-2800 **Austria** 43-1-877-8901 **Belgium** 32-9-385-5511 **Brazil** 5521-3237-9400
Canada 1-514-334-4372 **China** 86-21-64260808 **Czech Republic** 420-241-430-532 **Denmark** +45-4452-1000 **Finland** 358-9-804-22-00 **France** 33-1-47-95-60-00
Germany +49-(0)89-318-840 **Greece** 30-210-7774396 **Hong Kong** 852-2789-3300 **Hungary** +36-1-459-6100 **India** 91-124-4029300 **Israel** 972-3-9636050 **Italy** +39-02-216091 **Japan** 81-3-6361-7070 **Korea** 82-2-3473-4460 **Mexico** 52(55)5200-0520 **The Netherlands** +31-318-540666 **New Zealand** 64-9-415-2280 **Norway** 47-23-38-41-30 **Poland** 48-22-3319999 **Portugal** 351-21-472-7700 **Russia** 7-495-721-14-04 **Singapore** 65-6415-3188 **South Africa** 27-11-442-85-08 **Spain** 34-91-590-5200
Sweden 46-8-555-127-00 **Switzerland** 41-61-717-95-55 **Thailand** 662-651-8311 **United Kingdom** +44-(0)20-8328-2000