


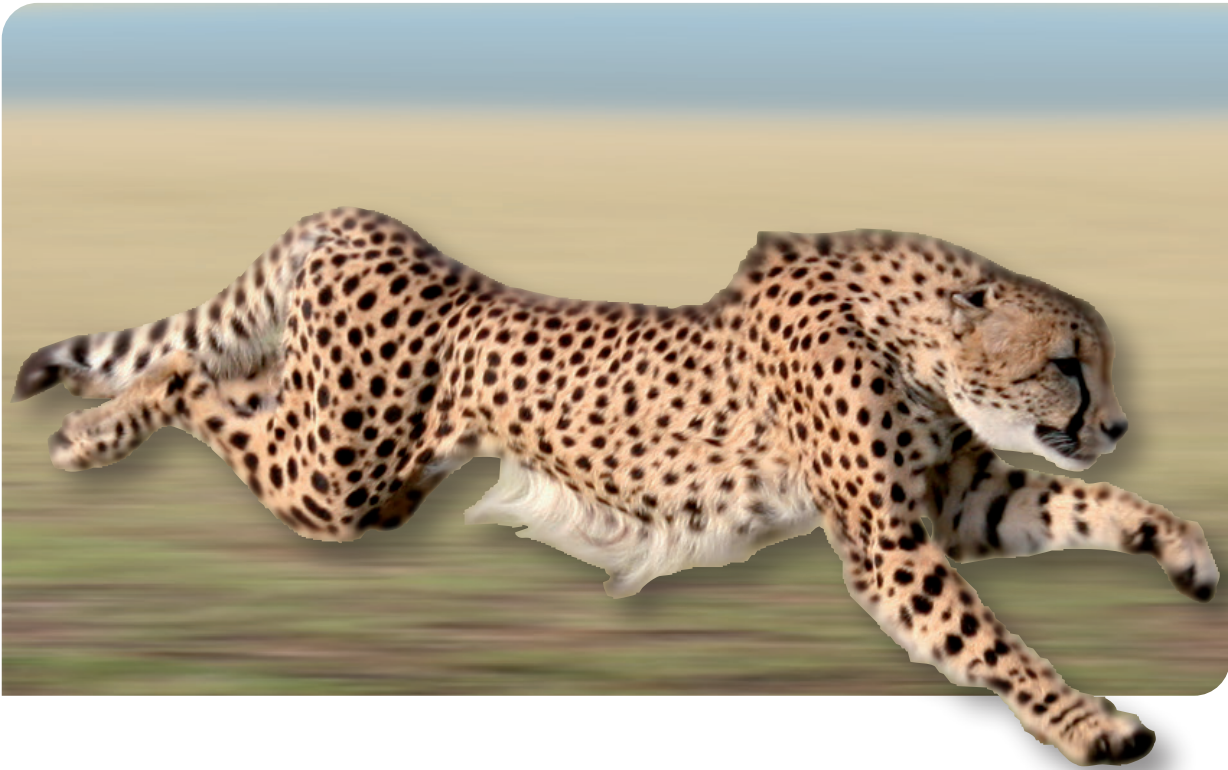

Genie Fast HIV 1/2 Assay

Performance with Convenience

- Detection of HIV-1 and HIV-2 antibodies
- Sample types: serum, plasma and whole blood
- Reliable, fast results in 10-30 minutes
- Easy transport and storage: 2-30°C

Listed on the WHO prequalified IVD products list

Visit www.bio-rad.com/diagnostics for more information.



BIO-RAD

Serum/Plasma Protocol

- Apply 80 µl or 3 drops of the plastic pipette (provided in kits #72330 and #72327) or using an automatic pipette




- Read results after 10-30 minutes




Fingerstick Whole Blood Protocol


- After using an antiseptic wipe and a sterile lancet hold the capillary tube 80 µl horizontally and touch the blood drop with the tip: capillary action will automatically draw the sample to the fill line and stop

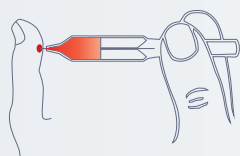

- Deposit 80 µl of fingerstick capillary blood in the circular deposit zone of the device

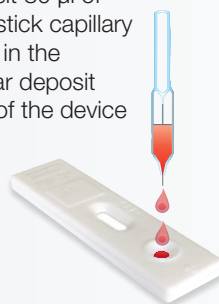

- Immediately add 2 drops of diluent

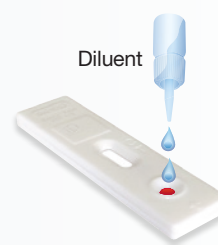

- Read results after 10-30 minutes



Results Interpretation



Ordering Information

Catalog #	Description
72327	Genie Fast HIV 1/2 Assay, 25 tests
72330	Genie Fast HIV 1/2 Assay, 50 tests
72347	Genie Fast HIV 1/2 with Fingerstick Consumables (Genie Fast cassettes + Alcohol swabs + Lancets + Capillary Microsafe pipettes), 25 tests

BIO-RAD is a trademark of Bio-Rad Laboratories, Inc.
GENIE and GEENIUS are trademarks of Bio-Rad Europe, GmbH
in certain jurisdictions.



**Bio-Rad
Laboratories, Inc.**

Clinical
Diagnostics Group

Website www.bio-rad.com/diagnostics **Australia** +61 (2) 9914 2800 **Austria** +43 (0) 1 877 89 01 9 **Belgium** +32 (0) 3 710 53 00 **Brazil** +55 11 3065 7550 **Canada** +1 514 334 4372 **China** +86 21 6169 8500 **Czech Republic** +420 241 431 660 **Denmark** +45 44 52 10 00 **Finland** +358 9 804 22 00 **France** +33 (0)1 47 95 60 00 **Germany** +49 (0) 89 31884 393 **Greece** +30 210 7774396 **Hong Kong** +85 2 2789 3300 **Hungary** +36 1 459 6190 **India** +91 124 4029300 **Israel** +972 03 963 6025 **Italy** +39 024 94 86 600 **Japan** +81 3 6361 7070 **Korea** +82 080 007 7373 **Mexico** +52 (55) 5488 7670 **The Netherlands** +31 (0)318 540 666 **New Zealand** +64 (9)415 2280 **Norway** +47 23 38 41 30 **Poland** +48 22 331 99 99 **Portugal** +351 21 47 27 700 **Russia** +7 495 721 1404 **Singapore** +65 6415 3170 **South Africa** +27 11 442 8508 **Spain** +34 91 490 6580 **Sweden** +46 844 98053 **Switzerland** +41 (0) 61 717 9555 **Taiwan** +886 (2) 2578-7189 **Thailand** (662) 651 8311 **United Kingdom** +44 (0)1923 471301